Troy Lee Designs®

KNEE BRACE

CATALYST X

 VQ OrthoCare[®]

CATALYST X

KNEE BRACE

- **MEDICAL GRADE KNEE BRACE**
- **TIBIAL DOCKING SYSTEM™** provides secure tibial control
- **MOLDABLE ALUMINUM FRAME** enables customization
- **AUTOCONFORM ALUMINUM-CORE WINGS** adapt to anatomy
- **INTEGRATED PCL STRAP** delivers increased stability
- **SIZING AND CE** woven label inside padded liner
- **OPEN MEDIAL UPPER AND LOWER SHELLS**
provide maximum clearance and comfort on the medial aspect of both legs, making it the perfect solution for patients engaging in activities including mountain biking and motocross
- **EXTENSION STOPS:** 0°, 5°, 10°, 15°, 20° and 25°
- **FLEXION STOPS:** 45°, 60°, 75° and 90°

MATERIALS:

FRAME: 100% Aluminum

STRAPS: 100% Nylon

SLEEVE: 15% Nylon 70% Neoprene 15% Volara Foam

PATELLA GUARD: 100% Polypropylene

RIVETS: 100% Stainless Steel

LINERS: 80% Nylon 20% Urethane Foam

EN

PROPER FITTING:

Measure your circumference 6" above mid-patella, mid-patella and 6" below mid-patella. They should measure out at approximately one of the following sizes:

SIZES AND CIRCUMFERENCE:

CATALYST X	S	M	L	XL	XXL
6" ABOVE MID-P	14.5"-17"	17"-19.5"	19.5"-22.5"	22.5"-25.5"	25.5"-30"
MID-PATELLA	11.5"-13"	13"-15"	15"-17"	17"-19"	19"-22"
6" BELOW MID-P	11.5"-13"	13"-14.5"	14.5"-16.5"	16.5"-18.5"	18.5"-22"

APPLICATION INSTRUCTIONS:

1- POSITION BRACE

- While sitting at edge of chair, flex knee to a 45° angle
- Align hinge centers with top of kneecap and position slightly behind midline of leg
- Ensure that adjustable tibial pad is centered on shin
- Confirm brace is not rotated on leg

2- FASTEN STRAPS

- While maintaining brace in position, fasten straps 1 -4 in sequence indicated on strap tabs. Straps should be comfortably tightened to properly secure brace.

Do not overtighten.

- Align hinge centers with top of kneecap and position slightly behind midline of leg.
- Ensure that adjustable tibial pad is centered on shin.
- Confirm brace is not rotated on leg.

- 1 Fasten Strap 1 snugly just above calf muscle. Strap 1 is important in maintaining brace position.
- 2 Fasten Strap 2 around back of lower thigh.
- 3 Fasten Strap 3 around calf muscle. Ensure all tibial padding is in contact with front of leg.
- 4 Fasten Strap 4 around back of upper thigh. Do not overtighten.
- 5 Adjust Strap 5 (PCL Strap) to ensure secure fit. PCL Strap typically requires minimal adjustment following initial fitting.

3- CHECK FIT

- Once straps are secure, hinge centers should be aligned with middle of kneecap and positioned slightly behind midline of leg.
- To ensure proper fit for activity, walk for several minutes. It is normal for brace to drop or settle into position. Make adjustments as necessary.

RANGE OF MOTION ADJUSTMENT:

Each Knee Brace comes with 5° extension stops preinstalled. At the direction of a medical professional, extension and flexion settings may be changed. Optional 0°, 10°, 15°, 20°, and 25° extension stops and 45°, 60°, 75°, and 90° flexion stops are included with each brace. **IMPORTANT: BOTH HINGES MUST BE SET TO SAME EXTENSION AND FLEXION SETTINGS. FOR 0° EXTENSION LIMITING, THE 0° EXTENSION STOPS MUST BE USED.**

INDICATIONS:

- Stabilization of ACL, MCL, PCL, LCL or treatment of combined ligament instabilities.
- Protection for surgical repairs.
- Conservative treatment of ligament damage or deficiency.
- Prophylactic bracing for athletic activities.

NOTICE:

While every effort has been made using state-of-the-art techniques to obtain maximum compatibility of function, strength, durability and comfort, this device is only one element on the overall treatment program administered by a medical professional. There is no guarantee that injury will be prevented through use of this device.

CAUTION:

Applicable federal, state, and/or local regulations may restrict this device to sale by or on the order of a practitioner licensed by law of the State in which he/she practices to use or order the use of this device.

EN

CLEANING INSTRUCTIONS:

Salt water, sand, dirt and other elements can cause damage. Thoroughly rinse brance with fresh water after exposure to these conditions. Towel or air dry. Liners should be removed and hand-washed as needed using cold water and mild detergent. Rinse thoroughly and air dry (do not machine-dry). Lubricate hinges with dry silicone spray as needed. Do not clean your protective gear with anything but soap and water. Do not use any abrasive cleaners or pads. Do not use hot water or chemical cleaners (like glass cleaner) or any solvents which contain petroleum based products like benzene or gasoline on your protective gear. Your protective gear can be weakened by these substances and may not provide its intended protection in an accident, which can result in serious injury or death.

This product meets the CE standard

EC

REP

Tested by EMERGO EUROPE - Molenstraat 15 2513 BH
The Hague The Netherlands

EN

WARNING!

Troy Lee Designs warrants all its products against defects in materials and workmanship for one year. Normal wear and tear, negligence, or improper care during the use of a product is not considered a defect. No warranty or representation is made as to this product's ability to protect user from any injury or death. The user assumes that risk. Protective Gear is built to assist in absorption of light impacts. It must be replaced after serious impact and should not be used if there are any cracks or broken plastic or torn material. Protective Gear is intended to minimize injuries, but it does not guarantee complete protection against injury or death. Do not modify this protective product in any way as it would dangerously reduce its intended performance. This product conforms to CE standards only in the configuration originally sold by the manufacturer. Troy Lee Designs does not offer any warranty or representation of this product's ability to protect the user if pieces are removed. Only the pieces in original positions conform to the CE standard. Only official TLD replacement parts are recommended.

NOTE: The life expectancy for Protective Gear is two years from date of purchase without incident of impact. Do not place gear in high heat and extreme cold conditions. This will damage the product.

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